SECUR TRACH

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REV. 01/2009

Medical Service
forniture ospedaliere

Via T. Caruto, 9 (Z.I.) 84131 Salerno (Italia)
Tel. +39-089-7724085 - Fax +39-089-303942 - e-mail: info@medserv.it

SECUR TRACH


The safe
percutaneous
tracheotomy


Medical Service
forniture ospedaliere

SECUR TRACH

Secur-Trach is an innovative system to perform percutaneous tracheotomies easily in a completely safe way. Secur-Trach is a modified endotracheal tube that has a high intensity L.E.D. for transillumination in its anterior distal wall and a safety shovel that extends in its distal posterior wall. The use of Secur-Trach allows, thanks to the transillumination technique, the immediate finding of the right needle insertion point and the safe insertion of needle and dilators while the standard ventilation continues. The shovel protects the posterior tracheal wall from any damage that needles and dilators may cause. Furthermore it facilitates the tracheotomic cannula insertion and the removal of the endotracheal tube itself.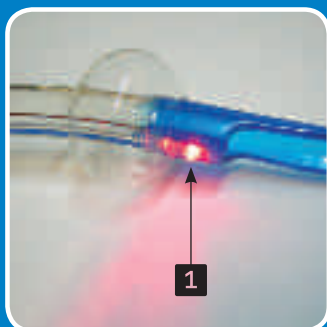

1 L.E.D. FOR
TRANSILLUMINATION
For the easy finding
of the right insertion point.

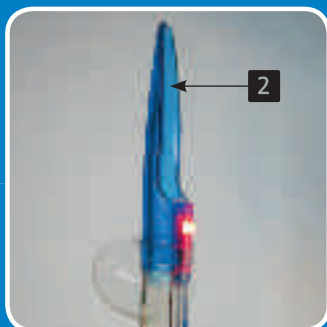2 SAFETY SHOVEL
For the protection of the
tracheal posterior wall.

The whole procedure is performed under STANDARD VENTILATION conditions without the need of re-positioning the endotracheal tube.

Secur-Trach is suitable for every percutaneous tracheotomy technique.

CE 0051

PATENT PENDING

ITEM	CODE
7mm cannula with normal balloon	DPST 0107
8mm cannula with normal balloon	DPST 0108
9mm cannula with normal balloon	DPST 0109
7mm cannula with off-set balloon	DPST 0207
8mm cannula with off-set balloon	DPST 0208
9mm cannula with off-set balloon	DPST 0209

Distributed by:

REV. 01/2009

Medical Service
forniture ospedaliere

Via T. Caruto, 9 (Z.I.) 84131 Salerno (Italy)
Tel. +39-089-7724085 - Fax +39-089-303942 - e-mail: info@medserv.it